# AIRTRAQ®

## OPTICAL LARYNGOSCOPE

US Patent No 6,843, 769

## INSTRUCTIONS FOR USE

ENGLISH Orotracheal intubation

## DESCRIPTION AND INDICATIONS

The AIRTRAQ is a SINGLE USE laryngoscope designed to facilitate intubation. It allows full visualization of the airway, during 100% of the intubation. It does not require hyperextension of the neck and permits intubating patients in virtually any position. It is provided clean, ready to use.

## SIZES

Choose the proper size AIRTRAQ for the ETT size being used.

## Regular: A-011

Blue. For use with ETT 7.0 – 8.5  
Minimum patient mouth opening: 18 mm

## Small: A-021

Green. For use with ETT 6.0 – 7.5  
Minimum patient mouth opening: 16 mm

## TECHNIQUE FOR USING THE AIRTRAQ

## I. PREPARATION

- Select the appropriate size AIRTRAQ based on the size ETT to be used.
- Turn on the light. Switch located on the side of the eyepiece. If no light is seen, replace unit.

**NOTE: The light stops blinking (30-60 sec) when the anti-fogging feature is fully activated.**

- Fully deflate the ETT cuff.
- Lubricate the ETT and place it into the lateral channel of the AIRTRAQ.
- Align the tip of the ETT with the end of the lateral channel.
- Appropriately lubricate the AIRTRAQ without contacting the lens.

## II. AIRTRAQ PLACEMENT

- Insert the AIRTRAQ into the midline of the patient's mouth. Take special care to avoid pushing the tongue inside the oropharynx. In some circumstances it may help to introduce the AIRTRAQ using the same technique as with a Guedel airway (Fig 1).
- Slide the AIRTRAQ through the oropharynx keeping it in the midline.
- Before the AIRTRAQ reaches the vertical plane, begin looking through the eyepiece to identify airway structures (Fig 2).
- Continue insertion until the epiglottis is identified. Place the tip of the AIRTRAQ in the vallecula.
- Gently lift up the AIRTRAQ to expose the vocal cords (Fig 3). Alternatively, the tip can be placed under the epiglottis, lifting it out of the way.

## III. ETT INSERTION

- **Align the vocal cords in the center of the visual field** by gently moving the tip of the AIRTRAQ as needed.
- Gently advance the ETT in the lateral channel until it is visualized passing through the vocal cords. Check insertion depth (Fig 4).
- Inflate the ETT cuff as normal and check for proper positioning. Reposition, if needed.

## IV. AIRTRAQ REMOVAL

- Separate the ETT from the AIRTRAQ by pulling it laterally from the ETT, while holding the ETT in position (Fig 5).

## WARNINGS AND PRECAUTIONS

- This product should only be used by personnel trained in insertion of endotracheal tubes.
- Do not sterilize
- Do not incinerate unless batteries have been removed.
- Do not submerge in liquids.
- Use only with non-flammable anesthetics.
- Do not put pressure on the teeth with this device.
- Do not force the AIRTRAQ into the upper airway

## USAGE TIPS

1. Initial experience should be gained in non-difficult airways.
2. Insert the AIRTRAQ, avoiding the tongue, and slide it softly and slowly.
3. Keep the AIRTRAQ in the mouth's midline.
4. Look through the eyepiece before the AIRTRAQ gets to the vertical plane.
5. If structures (arytenoids, epiglottis, etc.) are not clearly recognized, withdraw the AIRTRAQ slightly.
6. Once the tip is located at the epiglottis, either at the vallecula (Macintosh style), or under the epiglottis (Miller style), gently lift up the AIRTRAQ (do not tilt or use a lever action).
7. Advance the ETT slowly without twisting it.

## BATTERY REMOVAL (if required by facility protocol)

- Separate the eyepiece from the main body by pulling it up. Remove the battery cover by pulling it away from the main body (pull away from the small notches near the switch).
- Remove the alkaline batteries.

## WIRELESS/WIRED VIDEO SYSTEM ACCESSORY

The manufacturer offers a Clip-On wireless/wired Video System to allow viewing the intubation procedure on an external monitor. The Clip-On Video System is reusable and it is designed to be easily clipped on and off the AIRTRAQ unit. See Instructions for Use for the Clip-On Video System.

## MANUFACTURER'S WARRANTY

The manufacturer warrants the AIRTRAQ against faulty materials or manufacturing defects for only one use or until the expiration date, whichever comes first, provided that the AIRTRAQ is used in accordance with the procedures set forth in these instructions. This Warranty is applicable only if the device is purchased from an authorized distributor.

## The AIRTRAQ is designed for SINGLE-PATIENT-USE.

This device has not been designed to be cleaned or sterilized. Use beyond this recommendation may generate serious consequences in the product's performance and will void the AIRTRAQ's warranty. The manufacturer disclaims all other warranties, whether expressed or implied, including, without limitation, the warranties of merchantability or fitness for a particular use.

## ORAGE AND BATTERY SHELF LIFE

The AIRTRAQ should be stored at standard room temperature (0°C to 40°C) avoiding excessively humid atmospheres. Battery shelf life is limited to expiration date; dispose of them according to established recycling policies.

Manufactured by:  
Guangzhou Inimed Medical Appliance  
For Prodol Meditec (Spain)

For further advice on using the Airtraq please visit:  
<http://www.airtraq.com>  
or contact:  
[airtraq@prodolmed.com](mailto:airtraq@prodolmed.com)

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A AIRTRAQ é uma marca registada da Prodl Meditec S.A.